




You'll never have to press tablet blisters with your hands again.

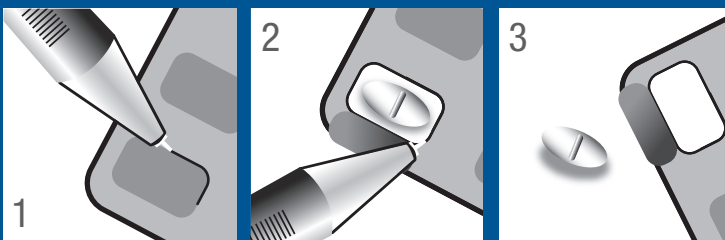

1 Turn the blister package and point the tip of the blister pen to the foil.

2 Cut through the tinfoil.

3 Turn the package and the tablet falls out.

Having trouble with your tablets?

- then the blister pen can help you

The blister pen will be a valuable tool for most patients suffering from arthritis.

The blister pen looks just like an ordinary ballpoint pen. It has no ink, though - the tip is pointed, enabling the pen to cut through the blister packaging's tinfoil quite easily.

When you've cut around the tablet, just turn the package and the tablet falls out.

The blister pen helps weak and sore arthritis suffering hands get tablets out of the blister packaging. The simple reason is that you no longer have to use physical strength pressing and squeezing the blisters.

rikacare.dk



DS/EN ISO 9001 CERTIFIED

RIKA PLAST AS, Saunte Bygade 12, 3100 Hornbaek, Denmark
Phone + 45 4975 0130 fax + 45 4975 0201, rika@rika.dk, rikacare.dk