



1 Remove the piston from the tube. Lower the wet cloth into the tube.



2 Use the weight of your body to press hard onto the piston. Press a few times..



3 Lift the Cloth-press and remove the cloth from the lower end of the tube.

## Cloth-press

- for weak hands

“Easy Squeezzy” enables weak and pain full hands to produce nice moist cloths for sweeping the kitchen.

- Use directly in your sink
- Easy to use for people with weak and aching hands
- Compact and unbreakable design
- Dishwasher safe

Having weak hands makes it difficult to squeeze a cloth properly - and without the sweep of a moist cloth, the kitchen often turn into a messy place. We are very happy to present “Easy Squeezzy”, a very practical and compact aid. Trials have confirmed that “Easy Squeezzy” is well accepted by rheumatic patients.